VAX-ID®

Exclusively for clinical investigation

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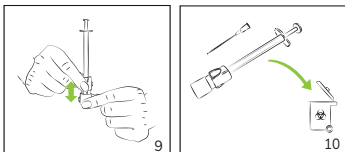
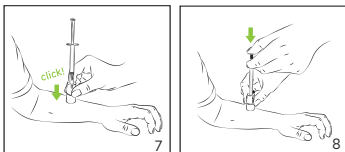
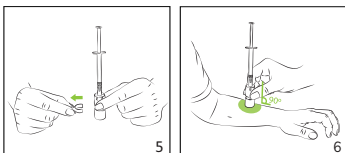
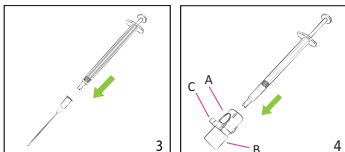
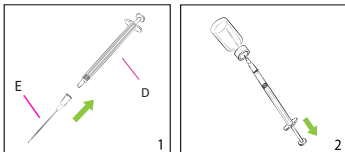
EN: User instructions

INTRADERMAL INJECTION ADAPTOR

VAX-ID® is a syringe adaptor for reliable, easy and accurate intradermal injection of medicinal products

Read all instructions prior to use

I. MATERIALS PROVIDED WITH EACH DEVICE

One adaptor made of:

A. Housing with injection needle

B. Foot

C. Pin

II. RECOMMENDED MATERIALS

D. 0.5 - 1.0mL syringe (Recommended: Low Dead Space)

E. Draw needle

WARNINGS AND PRECAUTIONS

Do not use if package is damaged or open

Single Use Only

Dispose of used device in accordance with applicable regulations

INSTRUCTIONS FOR USE

1. Attach the draw needle (E) to the syringe (D).
2. Draw the volume to be injected from the vial.
3. Remove the draw needle from the syringe.
4. Put the syringe into VAX-ID adaptor via the opening at the end of the housing (A). Make sure that the syringe tip is properly fitted into the Luer fitting of the embedded injection needle.
5. Activate VAX-ID by removing the pin (C).
6. Place the foot (B) on the skin at the dorsal side of the forearm, keeping the adaptor in a perpendicular position.
7. Push the housing (A) down to meet the foot (B); the needle will penetrate the skin.
8. Retain slight pressure with one hand on the housing (A) and use the other hand to inject the fluid by pushing the syringe plunger.
9. Remove VAX-ID from the skin and deactivate by pulling the foot (B) away from the housing.
10. Dispose of VAX-ID and the draw needle in a sharps container.

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