Do not use if package is damaged
Niet gebruiken indien de verpakking beschadigd is
Ne pas utiliser si l'emballage est endommagé
Nicht verwenden, wenn die Verpackung beschädigt ist
Non utilizzzare se la confezione è danneggiata
No lo utilice si el envase está dañado
Não usar se a embalagem estiver danificada
لا تستعمل التجهيز إذا كانت العبوة تالفة

Do not re-use
Niet herbruiken
Ne pas réutiliser
Nicht wiederverwenden
Non riutilizzare
No lo reutilice
Não reutilizar
لا تعد الاستعمال

Do not re-sterilize
Niet opnieuw steriliseren
Ne pas restériliser
Nicht reesterilisieren
Non risterilizzare
No reesterilizar
Não reesterilizar
لا تقم بإعادة التعقيم

Keep dry
Droog houden
Conservar au sec
Trocken halten
Conservare in un luogo asciutto
Manténgase en un lugar seco
Mantenha seco
أبقه جافاً

Manufacturer
Producent
Fabricant
Hersteller
Produttore
Fabricante
الشركة المُصنعة.

Manufacturing date
Productiedatum
Date de fabrication
Herstellungsdatum
Data di produzione
Fecha de fabricación
Data de fabricação
تاريخ التصنيع

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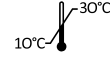
VAX-ID®

D00046 (32G)
D00048 (30G)
D00047 (27G)
D00037 (32G)
D00036 (30G)
D00035 (27G)

Exclusively for clinical investigation/Investigational use only (IUO)
For research use only (RUO)

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LANDOOR SRL
Via Copernico 38 Milan Italy



IDEVAX BV

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info@idevax.com www.idevax.com

VAX-ID is a registered trademark in the name of Idevax BV | VAX-ID is een geregistreerd handelsmerk van Idevax BV
VAX-ID est une marque déposée au nom d'Idevax BV | VAX-ID ist eine eingetragene Marke im Namen von Idevax BV
VAX-ID è un marchio registrato a nome di Idevax BV | VAX-ID es una marca comercial registrada en nombre de Idevax BV
VAX-ID é uma marca registrada no nome da Idevax BV | Idevax BV هو علامة تجارية مسجلة باسم Idevax BV

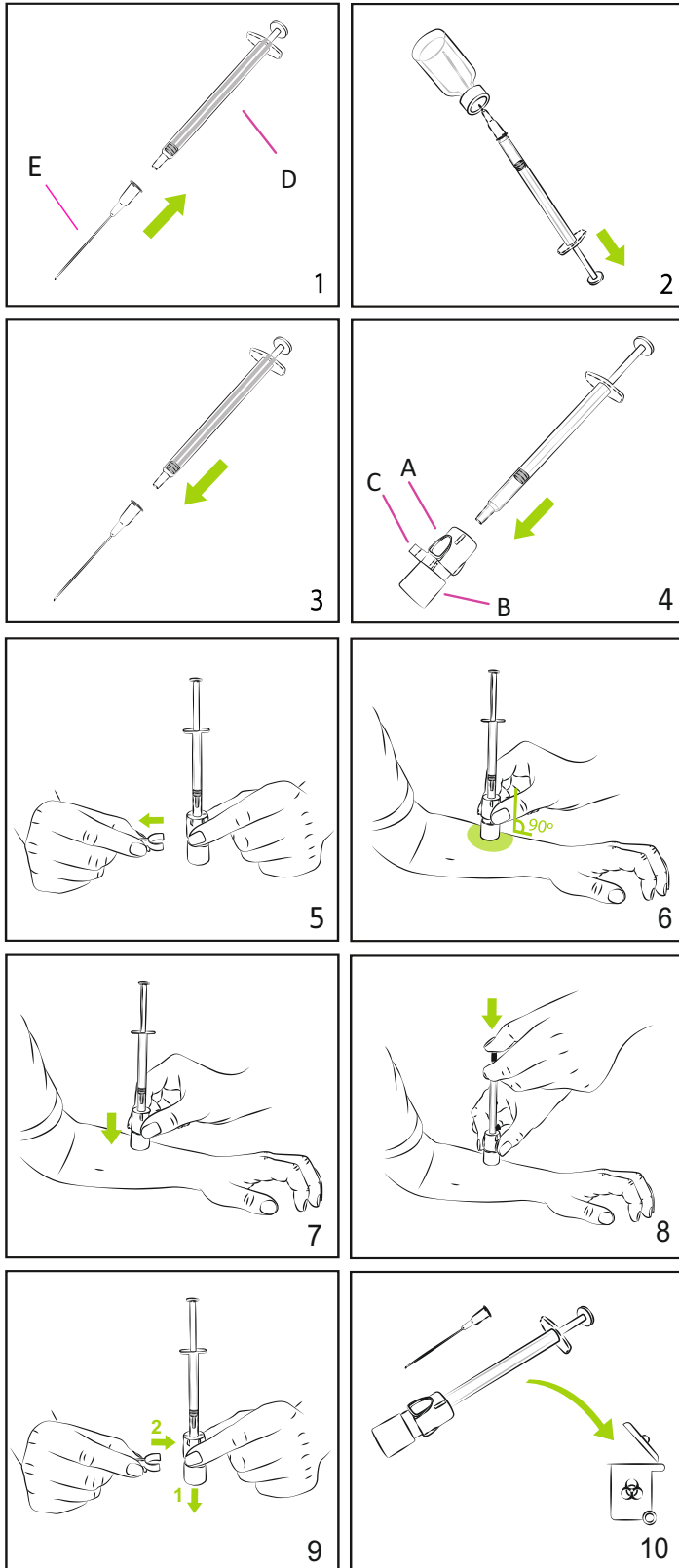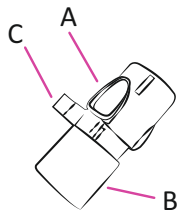IDEVAX™

EN: User instructions

DESCRIPTION & INDICATIONS

Intradermal injection adaptor: VAX-ID® is a syringe adaptor for easy and reliable intradermal injection of medicinal products. Read all instructions prior to use.

INTENDED USER/ USER QUALIFICATIONS

Healthcare professional (E.g. nurse, doctor, pharmacist)

TARGET POPULATION

All ages and populations.

MATERIALS PROVIDED WITH EACH DEVICE

One adaptor made of:

- A. Housing with injection needle
- B. Foot
- C. Pin

RECOMMENDED MATERIALS

- D. 1.0 mL Luer slip fit syringe (Recommended: Low Dead Space)
- E. Draw needle

WARNINGS AND PRECAUTIONS

Do not use if package is damaged or open

Single Use Only. Re-use may lead to contamination by foreign material/organisms thus infection and/or illness.

Dispose of used device in a sharp's container in accordance with applicable national regulations.

Any serious incidence (as defined per the European Medical Device Regulation 2017/745) should be reported to the manufacturer and the competent Authority.

ADVERSE EVENTS

Intradermal injections may be associated with transient local skin reactions including local pain, edema, erythema, itching, hardening, peeling, discoloration at the injection site, most likely related to the injected substance, and not the device.

Adverse events should be reported to the manufacturer.

CONTRAINDICATIONS



Do not use on wounds, scars, nor damaged skin.

STORAGE

The device should be stored at temperatures ranging from 10-30°C, in a dry environment, with no direct sunlight, and without large temperature fluctuations.

INSTRUCTIONS FOR USE

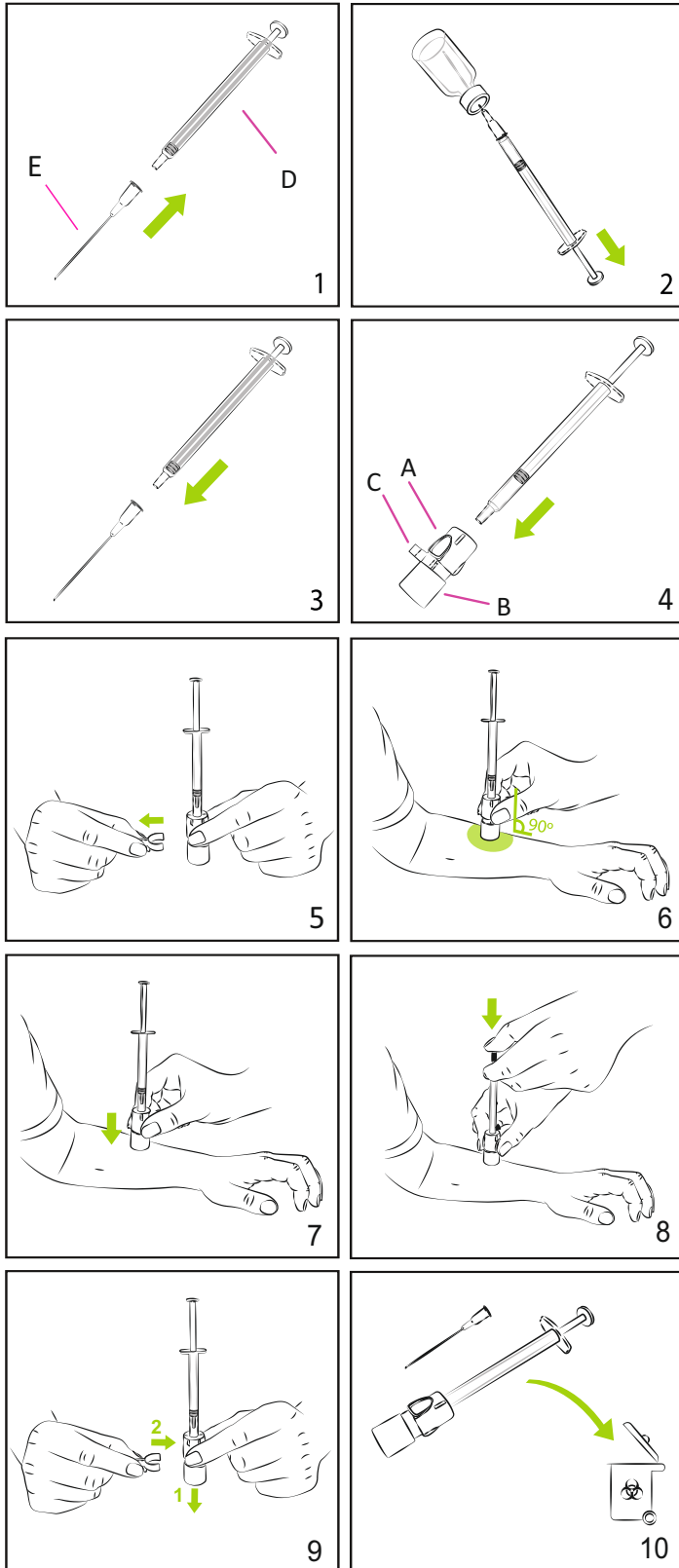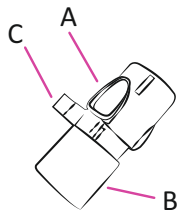1. Attach the draw needle (E) to the syringe (D).
2. Draw the volume to be injected from the vial.
3. Remove the draw needle from the syringe.
4. Put the syringe into VAX-ID® adaptor via the opening on the end of the housing (A). Make sure that the syringe tip is properly fitted into the Luer fitting of the embedded injection needle.
5. Activate VAX-ID® by removing the pin (C).
6. Place the foot (B) on the skin, keeping the adaptor in a perpendicular position.
7. Push the housing (A) down to meet the foot (B): the needle will penetrate the skin.
8. Retain slight pressure with two fingers on the housing (A) and use another finger to slowly (3-5 seconds) inject the fluid by pushing the syringe plunger, expect resistance while injecting, and wait few (2-3) seconds before removing the VAX-ID®.
9. Remove VAX-ID® from the skin and deactivate the device by pulling the foot (B) away from the housing (A) and placing the pin (C) back in its original position.
10. Dispose VAX-ID® and the draw needle in a sharps container.

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9. Rimuovere VAX-ID® dalla pelle e disattivarlo allontanando la base (B) dall'alloggiamento (A) e rimettendo il perno © nella sua posizione originale.