



PRODUCT INFORMATION

Product	VAX-ID® 2200
Description	Intradermal injection adaptor for easy, accurate, and standardized injection of medicinal products in the skin. The adaptor is single use and can be provided sterile and non-sterile (to be sterilized before use). It can be preconfigured with different needle gauges and allow for a predefined penetration depth.
Classification	Class IIa medical device determined by Rule 6 for a surgically invasive device intended for transient use.
Materials	Polypropylene (PP) and Stainless Steel (needle)
Dimensions	40 x 29 x 17 mm (Device) 150 x 75 x 17 mm (Pouch)
Weight	± 4 grams
Intended Purpose	VAX-ID® is an injection adaptor for easy and reliable intradermal injection of medicinal products at a predefined penetration depth.
Intended User	Professional medical care trained personnel (e.g. nurse, doctor, pharmacist...).
Intended Population	All ages and populations.
Indications for use	Intradermal injection of approved medical substances.
Contra-indications	Do not use on wounds, scars, nor damaged skin.

DEVICE CONFIGURATIONS 5430003573000022000000UC

Needle Gauge	REF		Average Injection Depth	Needle Exposure Length
32G	D00046	VAX-ID® 2200 32G sterile	0.5mm	0.85mm
30G	D00048	VAX-ID® 2200 30G sterile	0.6mm	1.15mm
27G	D00047	VAX-ID® 2200 27G sterile	0.8mm	1.55mm

Average dead space of VAX-ID®: 14µL

VAX-ID® has a validated sharps injury protection system according to ISO 23908:2024

VAX-ID® IS TO BE USED IN COMBINATION WITH THESE MEDICAL DEVICES

Draw Needle	Commercially available draw needle
Low Dead Space syringe with Luer Slip tip	Commercially available 1 mL syringe with Luer Slip tip like HSW Soft-Ject, Low Dead Space or B.Braun Injekt®-F Luer Solo




PACKAGING, STORAGE, AND SHELF-LIFE

Packaging	Boxes of 100 devices
Materials	See-through pouch of medical grade PET/PP foil and paper
Storage	Recommended temperature: 10-30°C Dry environment No direct sunlight Without large temperature fluctuations
Shelf-life	5 years
Warnings	Do not use if package is damaged or open. Single Use Only. Re-use may lead to contamination by foreign material/organisms thus infection and/or illness. Dispose of used device in a sharp's container in accordance with applicable national regulations. Any serious incidence (as defined per the European Medical Device Regulation 2017/745) should be reported to the manufacturer and the competent Authority.



INSTRUCTIONS FOR USE

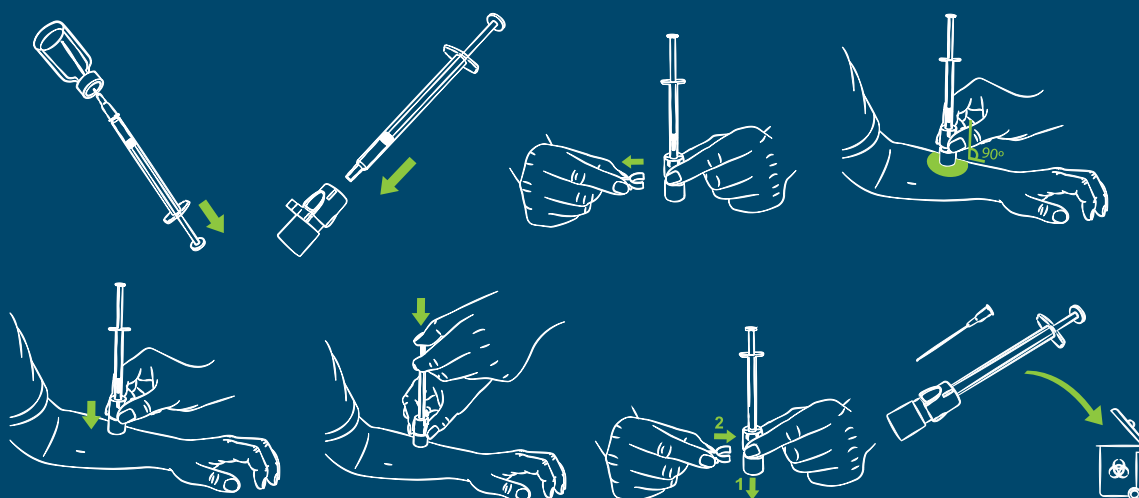

VAX-ID® 2200 in pouch